Technische Information / Technical information / Informations techniques /  
Información técnica / Informazioni tecniche / Teknisk information / Техническа  
информация / Technische informatie / Specyfikacja / Τεχνικές πληροφορίες  
/ Technické informace / Technikai információ / Teknik Bilgiler / Техническая  
информация

**4W4021, 4W4021-TF, 4W4121, 4W4022, 4W4122, 4W4062,  
4W4162, 4U4010, 4U4010-TF, 4U4110, 4U4021,  
4U4021-TF, 4U4121, 4U4022, 4U4122, 4U4062, 4U4162,  
4L4021, 4L4121, 4L4022, 4L4122, 4L4062, 4L4162,  
4U4065, 4U4165, 4U4025, 4U4125**

<b>DE - LITE LINER® Silikon mit Stoffbezug</b>	<b>2</b>
<b>GB - LITE LINER® Silicone with fabric cover</b>	<b>9</b>
<b>FR - LITE liner® silicone avec couverture de tissu</b>	<b>16</b>
<b>ES - LITE LINER® Silicona con cubierta de tela</b>	<b>23</b>
<b>IT - LITE LINER® silicone con copertura in tessuto</b>	<b>30</b>
<b>SV - LITE LINER® -silikon med tygskydd</b>	<b>38</b>
<b>BG - LITE LINER® силикон с капак на плат</b>	<b>44</b>
<b>NL - LITE LINER® Silicone met stofomslag</b>	<b>51</b>
<b>PL - Silikon lite Liner® z pokrywą tkanin</b>	<b>58</b>
<b>EL - Silicone LITE LINER® με κάλυμμα υφάσματος</b>	<b>65</b>
<b>CS - Silikon LITE LINER® s krytem tkaniny</b>	<b>72</b>
<b>HU - LITE LINER® szilikon szövet burkolattal</b>	<b>78</b>
<b>TR - Kumaş kapağılı LITE LINER® Silikon</b>	<b>85</b>
<b>RU - LITE LINER® Силикон с тканевой крышкой</b>	<b>92</b>

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# GB LITE LINER® Silicone with fabric cover

**MD**

## Information

- Read this document carefully before using the product. Follow the safety instructions to avoid injury and product damage.
- Instruct the user in the proper and safe use of the product.
- Reporting a serious incident: Any serious incident related to the product must be reported to the manufacturer and the competent authorities.
- Keep this document in a safe place.

## 1. Description

The product is a silicone liner that covers the residual limb and provides a contact surface between the residual limb and the socket of the prosthesis. It is available as a locking liner with a distal connection option or as a cushion liner without a distal connection option.

A locking liner has either a fixed or a removable distal connector.

A cushion liner does not have a distal port and must be used with a sleeve to hold the vacuum.

## 2. Indicated Use

The silicone liners are to be used exclusively for the prosthetic fitting of the lower extremities. The product is the interface of the residual limb with the prosthetic socket.

Suitability of the device for the prosthesis and the patient must be evaluated by a healthcare professional.

The device must be fitted and adjusted by a healthcare professional.

It fulfills the following functions:

- The prosthesis is attached to the limb stump using pin locking or vacuum system technology
- residual limb compression (prevention of edema, reduction of residual limb volume fluctuations)
- Protection and care of the skin
- Shock absorption
- Cushioning

## 2.1 Indication

The silicone liners are only to be used for prosthetic restoration after amputation and/or congenital deficiency of the lower extremities.

## 2.2 Contraindication

- Too many soft tissues or soft tissue overhangs
- Unstable knee joint
- Stump load impossible
- Bad hygiene
- Very sensitive neuromas
- Allergy / intolerance to the material
- Contact with irritated / sick skin
- Contact with open wounds

### Warning!

If you have severe diabetes and poor blood circulation, consult a doctor before use

### 2.3 Environmental conditions

Storage and transportation

Do not expose the product to temperatures above +55 °C and below -20 °C.

Items must be stored at room temperature in a dry place out of direct sunlight if left unused for a long time.

### 3. Technical data

Name	Art. No.	Activity Level	TT / TF
LITE LINER Silikon SkinTex 3 L	4L4021-10, 4L4121	2-4	TT
LITE LINER Silikon SkinTex 3 L SOFT	4L4022-10, 4L4122	1-3	TT
LITE LINER Silikon SkinTex 6 L SOFT	4L4062-10, 4L4162	1-3	TT
LITE LINER Silikon SkinTex 1 U	4U4010-10, 4U4110	2-4	TT
LITE LINER Silikon SkinTex 2U	4U4021-10, 4U4121	2-4	TT
LITE LINER Silikon SkinTex 2 U SOFT	4U4022-10, 4U4122	1-3	TT
LITE LINER Silikon SkinTex 6 U SOFT	4U4062-10, 4U4162	1-3	TT
LITE LINER Silikon SkinTex 2	4W4021-00, 4W4021-05, 4W4021-10, 4W4021-15, 4W4121	2-4	TT
LITE LINER Silikon SkinTex 2 SOFT	4W4022-00, 4W4022-05, 4W4022-10, 4W4022-15, 4W4122	1-3	TT
LITE LINER Silikon SkinTex 6 SOFT	4W4062-00, 4W4062-05, 4W4062-10, 4W4062-15, 4W4162	1-3	TT
LITE LINER Silikon SkinTex 2 U CARE	4U4025-10, 4U4125	1-3	TT
LITE LINER Silikon SkinTex 6 U CARE	4U4065-10, 4U4165	1-3	TT

LITE LINER SkinTex 1 U TF	4U4010-TF	2-4	TF
LITE LINER Silikon SkinTex 2 TF	4W4021-TF	1-3	TF

### 3.1 Service life

The product is a wearing part that is subject to normal wear and tear. After a period of use of 6 months, we recommend replacing the liner.

### 3.2 Restriction of Use

Due to the adjustment, which must be carried out by trained specialists, and for reasons of hygiene, we would like to point out that this product is only to be used by one person.

### 3.3 Misuse

The product is intended exclusively for prosthetic fitting in accordance with its intended use and is not suitable for other sports activities.

### 3.4 Unallowable combination of prosthetic components

Risk of injury due to breakage or deformation of the product

- Only combine the product with prosthetic components that are approved for that purpose.
- Based on the instructions for use of the prosthetic components, verify that they may be combined with each other.

### 3.5 Use under unallowable environmental conditions

Risk of injury due to damage to the product

- Do not expose the product to unallowable environmental conditions.
- If the product has been exposed to unallowable environmental conditions, check it for damage.
- If damage is apparent or in case of doubt, do not continue using the product.
- Take suitable measures if required (e.g. cleaning, repair, replacement, inspection by the manufacturer or a specialist workshop etc.).

#### 4. General safety instructions

**Warning!**

Keep the product safe from children - There is a risk of suffocation if the product is pulled over the mouth and nose!

Take care not to wear the liner on injured skin!

**Warning!**

The silicone liner should not be worn directly on open wounds or broken skin, or used by people prone to skin inflammation.

**Warning!**

- Upon receipt and at regular intervals, please check the edge of your silicone liner for any tears or irregularities. Such defects in the edge area can lead to cracks in the material and damage to your silicone socket. Improper use or use of a defective product can lead to potential adhesion failure. If such defects occur, the user must stop using the product and must consult an orthopedic technician.
- To avoid the risk of personal injury and product damage, do not use the product beyond the tested service life.
- Before use, an individual adjustment of the component and the prosthesis in which this component is used is required.
- Before use, instruction in the use of the component by an experienced orthopedic technician is required.
- Incorrect selection, adjustment, use and/or inadequate control can lead to damage to health.
- The prosthesis and its functional components must be checked regularly by an orthopedic technician for function, possible wear and tear and possible damage.
- In connection with the use of prosthesis components for the production of external limb prostheses, the following must be observed: Use components only for their intended purpose.
- If components with different maximum loads are combined, the maximum load of the weakest component applies to the entire prosthesis.
- If components for different activity levels are combined, the activity level of the component with the lowest activity applies to the entire prosthesis.
- The use of tested individual components with the CE mark does not release the technician from his obligation to check the component combination for its usefulness, proper assembly and safety within the scope of his possibilities.
- If there are indications that a component combination does not meet the required level of safety, the components must not be combined. The prosthesis must be constructed in accordance with the generally recognized professional rules of the orthopedic technician trade.
- The tightening torques specified for the component must be observed. A suitable tool (torque wrench) must be used for this.
- Before handing them over to the user, the fitting parts must be secured against unintentional loosening or twisting of the screw connections with a suitable screw locking device.
- Safety-relevant regulations for individual components (e.g. special combination options, maintenance intervals, etc.) must be observed.
- When combining different components (from one or different manufacturers), the following applies: Only use components that meet the requirements of DIN EN ISO 10328 and the requirements of the Medical Devices Act.

## 5. Size selection

### 5.1 How to choose the correct liner size

- Measure the circumference of the residual limb at 4 cm from the distal end. With a transtibial liner, the knee should be slightly bent to allow the soft tissue to hang down.
- Choose the liner size corresponding to this measurement or the next smaller size, respectively.
- Do not choose a liner whose size exceeds the measurement taken.
- Never reduce by more than one liner size in order to avoid unnecessarily high compression.

#### Notice!

A liner with a connector often has a matrix with a length of 10 cm integrated. This matrix should end below the head of the fibula and must not exceed it, as this could result in problems during knee flexion.

The selection of the correct liner size is of crucial importance for comfort and safety.

- If the liner is too tight, this could result in pistoning, numbness and blisters.
- Is the liner too loose, this may cause increased perspiration and rotation of the stump inside the liner which could lead to blisters and skin rashes.
- Please contact your prosthetist if one of the symptoms or other problems should occur.

### 6. Adjustment of the liner

Threaded liners usually have a matrix. The matrix prevents excessive elongation of the liner, with little restriction of radial extensibility. The length of the matrix depends on the application area of the liner. Liners with a matrix in the distal area are particularly suitable for TT prostheses. Liners with a full-length matrix are commonly used for TF prostheses.

#### Caution!

The product must be thoroughly inspected for defects before fitting.

### 6.1 Trimming of a liner

#### Caution:

Do not follow socket edge during trimming, but always cut all around at an equal height slightly above/beyond the socket edge. Do not use zig-zack scissors in order to avoid a possible tearing of the heavily pre-stressed liner at the serrated areas during trimming.

Do not trim the liner below the edge of the matrix or below the proximal stem cut lines. Excessive trimming may reduce adhesion between the product and the limb, thereby affecting fixation.

### 6.2 Adjustment of the socket

Proper functioning of the product depends to a large extent on the fit of the upper. The following points should be checked in order to achieve the highest level of adhesion, control and comfort.

- The patient can easily put on the socket without the use of lubricants.
- Complete contact between the liner and the inner wall of the socket.
- No development of air pockets when the patient puts on the prosthesis.
- Allows full movement without compromising distal adhesion.

#### Caution!

If user feedback indicates that one or more of the above conditions are not met, the socket must be redesigned or remanufactured accordingly.

#### Caution!

When using a cushion liner, the socket must be airtight to ensure vacuum fixation when a sleeve is applied.

### 6.3 Optional: Installing the pin

The pin for connecting the liner to the shuttle lock is screwed into the thread at the distal end of the liner. Apply medium-strength Loctite to the thread beforehand.

Screw the pin into the thread and tighten carefully with 3-4 Nm.

## 7. Description

### 7.1 Donning of the liner

#### Warning!

Cover open wounds or broken skin with a plaster or other appropriate wound dressing to prevent direct contact between the wound and the liner.

1. Turn the silicone liner inside out, expose the distal end of the liner as much as possible and place it against the stump by avoiding any air pockets.
2. Take care that the screw thread is centered and in line with the stump.
3. Then roll the liner on the stump carefully. To avoid unpleasant wrinkles in the area of the hollow of the knee, carefully pull the socket up behind the knee and put it on the skin again in a stretched state.  
Take special care that the socket will not be damaged and that the pin receptacle is positioned as central as possible.
4. Please avoid to pull at the upper end of the liner as this results in an increased tension on the liner brim and the subjacent skin. This manifests itself in water blisters, weals or similar symptoms. If wrinkles should have occurred during donning of the liner, please repeat procedure.

#### Notice:

In the case of silicone liners, it is advisable to treat the skin in areas of high friction with skin protection lotion 4W296 or selectively with Protect-Gel 4W293 before putting on the liner.

#### Warning:

A large-area treatment with Protect-Gel 4W293 may cause a losing of the liner. The application of Protect-Gel 4W293 is intended only for small areas, i.e. as big as a coin. We, therefore, implicitly recommend to apply the corresponding care products at least 10 minutes before donning of the liner.

#### Caution!

At a fitting with a locking system, it is absolutely necessary to ensure that the patient has a socket with distal end contact. Always place the locking system following the longitudinal axis of the stump.

## 7.2 Donning the socket

### Cushion liner:

Use this in conjunction with a knee cap. Refer to the knee cap user manual for instructions on correct use.

Using your body weight, slide into the socket until the residual limb is firmly seated before using the knee cap to establish fixation in the Vacuum System technique.

### Locking liner:

Using body weight, slide into the shaft until the distal connector firmly engages the detent and a secure connection is made.

#### Caution:

Make sure the connection between the distal connector and the detent is secure and free of play.

## 8. Cleaning and care

### 8.1 Care of the liner

Clean the product daily after use and before first use.

- Turn the liner inside out.
- Hand wash the liner in warm water (30°C) with neutral soap. Only use water and soap for this, but no other harsh cleaning liquids, alternatively we recommend the cleaning lotion 4W295.
- Turn the liner over again so that the outside/textile is on the outside.
- Tuck a towel into the liner and lay the liner flat to dry on a drying rack. Alternatively, use a drying stand.
- The product can also be machine washed with a mild detergent and a low spin speed (40 °C). Fabric softeners, bleach and other products/cleaning solutions can damage the product and must not be used
- You should not store your socket unwashed or twisted. A spare socket is very useful in this situation and gives additional security.
- Liners that have already been worn usually feel softer than new liners.

**Notice:**

Using the wrong detergent may damage the product.

Do not use bleach, harsh chemicals or objects such as salt water, acids, dust or sand as this may damage the fabric on the outside of the liner or the silicone surface.

**Notice!**

Dry the liner textile side out to prevent damage.

**Caution!**

When handling your silicone socket, be careful not to damage it with your fingernails or sharp objects such as the locking pin.

**Caution!**

Please also avoid contact of the liner with glass or carbon fibers or other foreign particles, as these materials cannot be removed by normal cleaning. Any residue left on the silicone can cause skin irritation. If the liner has come into contact with such materials, it should be returned to the prosthetist.

**8.2 Care of the stump**

At the beginning of your fitting with a silicone liner, you may experience increased perspiration. However, this will decrease significantly after one to two weeks, since, in contrast to more conventional fittings, there is hardly any friction between the skin and the socket. With increased physical exertion, sweating can still occur. In the case of sensitive skin, skin irritation can occasionally occur. In this case, consult the medical supply store that carried out your treatment or your doctor. Such skin irritation is caused, for example, by the slight pull on the hair, which can irritate the hair roots.

The stump must be thoroughly washed daily with neutral soap. We recommend using a mild, pH-neutral, fragrance-free and dye-free soap. For dry skin, apply a pH-balanced, fragrance-free and dye-free lotion to condition and soften the skin.

**9. Additional products**

The following accessories can be used in combination with cushion and locking liners.

**9.1 Use of Distal Cups 4W2000**

The Distal Cups 4W2000 offer increased comfort for conical stumps with small soft tissue covering. They are used in combination with silicone liners in order to compensate conical stump shapes and provide an additional padding.

The Silicone Distal Cups 4W2000 adapt to the stump and, consequently, decrease pressure over sensible bony areas during walking.

**9.2 Use in combination with Liner Pad 4W15**

If there should be voids in the distal area of the liner, caused by e.g. scars in the stump or temporarily occurring weight fluctuations which appear in this area, so these voids can be filled in with the Liner Pad 4W15. For this, that silicone pad corresponding to the liner size is chosen and is carefully placed inside the liner before donning.

**Caution!**

Please ask your prosthetist to demonstrate this procedure exactly in order to avoid an improper or careless application which may result in skin irritations or even in hematomas.

**9.3 Use of stump socks**

Any fluctuations in volume of the residual limb that may occur should be compensated for if there is a noticeable loosening of the seat in the prosthesis socket.

Stump socks with a hole for the pin are available for liner variants with a PIN connection.

For cushion liner variants, we recommend stump socks without a distal hole.

**Notice!**

To protect the silicone liner from possible damage caused by friction, a thin residual limb stocking with a distal opening for the pin should also be pulled over the silicone liner. This reduces the friction between the socket and the liner.

## 10. Maintenance/Service

The Silicone liner is subject to natural wear and tear during continuous use and must therefore be checked at regular intervals and replaced if necessary.

### **CAUTION: Failure to follow maintenance instructions**

Risk of injury due to a change in function or loss of function as well as damage to the product

Observe the following maintenance instructions:

- Perform a visual and functional check of the prosthesis components after the first 30 days of use.
- Check the complete prosthesis for wear during the normal consultation.
- Carry out annual safety checks.

## 11. Guarantee

The following guarantee declaration applies in addition to any existing legal warranty claims and is independent of these.

### 11.1 Conditions of entitlement

The manufacturer provides a warranty on the product from the date of purchase.

A guarantee can only be given if the product is used under the specified conditions and for the intended purposes. The warranty covers defects that are demonstrably due to material, manufacturing or construction errors and that are claimed against the manufacturer within the warranty period. Claims under the manufacturer's warranty can only be made if the product concerned has been installed or adapted by or under the supervision of trained specialist personnel. Maintenance intervals, if any, must have been demonstrably carried out.

### 11.2 Scope and exclusion of warranty

The product is guaranteed to be free of material, manufacturing or construction defects. The manufacturer's warranty does not cover defects that are attributable :

- to incorrect installation
- on operating errors
- to improper / improper use
- on normal wear and tear
- on improperly machined or sharp-edged edges of the socket
- external influences such as force majeure or third party intervention

### 11.3 Duration of warranty

The duration of the manufacturer's warranty varies depending on the product. In any case, the duration of the manufacturer's warranty starts from the date of delivery to the customer. If used properly, the fabric coated LITE LINER Silicone is guaranteed for 6 months.

### 11.4 Liability

The manufacturer is only liable if the product is used in accordance with the descriptions and instructions in this document. The manufacturer is not liable for damage caused by non-compliance with this document, in particular by improper use or unauthorized modification of the product. Each manufacturer can only be held liable for the failure of its own fitting parts. A further liability of the manufacturer is only possible if his own components can be proven to have been the cause of the damage or functional failure of components from other manufacturers.

### 11.5 Warranty service

If all conditions for a warranty claim are met, Wagner Polymertechnik GmbH will, at its discretion, repair the damaged product, send a replacement free of charge or issue a credit note for the cost of the product.

## 12. Declaration of Conformity

As the manufacturer, Wagner Polymertechnik GmbH declares under sole responsibility that the LITE LINER® Silicone with fabric cover comply with the requirements of Regulation (EU) 2017/745 for medical devices (MDR).

## 13. Disposal

The product is to be recycled or disposed of without risk to others or the environment. Please observe the applicable national regulations.

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