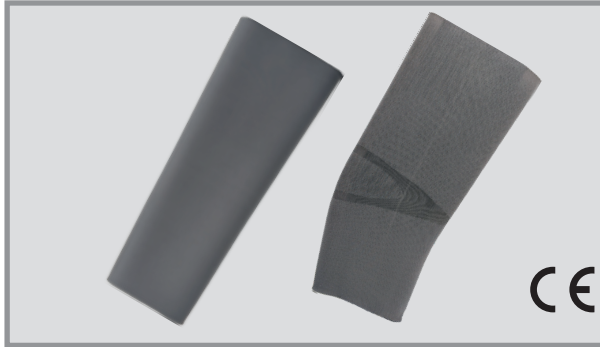
Technische Information
Technical Information
Informations techniques
Información técnica

4W80/4W82/4W84/4W85/4W87/4W88/4W89

DE - LITE LINER® Gel Kniekappe mit Stoffbezug	2
GB - LITE LINER® Gel sleeve with fabric cover	6
FR - Manche en gel LITE LINER® avec couvercle en tissu	10
ES - Manga de gel LITE LINER® con cubierta de tela	14
IT - LITE LINER® Gel Sleeve con copertura in tessuto	18
SV - LITE LINER® gelhylsa med tygskydd	22
BG - LITE LINER® гел ръкав с капак на плат	25
NL - LITE LINER® gelhoes met stofomslag	30
PL - Żelowy rękawa LITE LINER® z pokrywą tkaniny	34
EL - LITE LINER® Gel Sleeve με κάλυμμα υφάσματος	38
CS - LITE LINER® Gel Smeevé s krytem tkaniny	42
HU - LITE LINER® gélhüvely szövetborítással	46
TR - Kumaş örtüsü ile LITE LINER® Jel Sleeve	50
RU - LITE LINER® гелевой рукав с тканевой крышкой	53

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GB LITE LINER® Gel sleeve with fabric cover

MD

Information

- Read this document carefully before using the product. Follow the safety instructions to avoid injury and product damage.
- Instruct the user in the proper and safe use of the product.
- Reporting a serious incident: Any serious incident related to the product must be reported to the manufacturer and the competent authorities.
- Keep this document in a safe place.

1. Purpose of use

The TPE knee caps are to be used exclusively for the prosthetic fitting of the lower extremities.

2. Mode of action

The knee sleeves are coated on the inside with copolymer gel. The TPE knee sleeves are designed to prevent the residual limb from slipping out of the prosthesis' socket. The TPE knee sleeves are also used for sealing in vacuum socket systems.

2.1 Indication

The TPE knee sleeves can be used as knee support for transtibial amputees or as sealing caps for vacuum sockets. They can also be used as an additional knee guide and stabilisation for short prostheses and smaller sizes in arm prosthetics as a support bandage.

2.2 Contraindication

The TPE knee sleeves must not be used in the case of

- Material incompatibility/allergy
- Injured, irritated or damaged skin - cover open wounds!
- Severe circulatory problems.

Attention!

If you have severe diabetes and poor blood circulation, consult a doctor before use.

2.3 Environmental conditions

Storage and transportation

Do not expose the product to temperatures above +55 °C and below -20 °C.

Items must be stored at room temperature in a dry place out of direct sunlight in the original packaging if left unused for a long time.

3. Technical data

4W80/4W82/4W85:

Length: approx. 35 cm

Wall thickness and shape: simple, conical design; therefore flexibly positionable

Material: soft TPE gel with medical white oil, Shore 00: approx. 25 - 30

Abrasion-resistant cover made of polyamide / elastane

4W84:

Length: approx. 38 cm

Wall thickness and shape: simple, conical design; therefore flexibly positionable

Material: soft TPE gel with medical white oil, Shore 00: approx. 25 - 30

Abrasion-resistant cover made of polyamide / elastane

4W87/4W88/4W89:

Length: approx. 40 cm

Wall thickness and shape: conical design with additional patella and approx. 15 ° preflexion; therefore particularly high freedom of movement in the knee area

Material: soft TPE gel with medical white oil, Shore 00: approx. 25 - 30

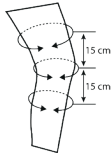
Abrasion-resistant cover made of polyamide / elastane

3.1 Sizing Chart

3.1.1 4W80/4W82

Measure the circumference of the residual limb about 15 cm above and below the kneecap and choose the correct size

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
3.1.2 4W84

Measure circumference approx. 15 cm above and below mid-patella and choose the correct size.

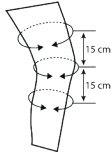Total Length: 380 mm


Art. No.	Lower Circumference	Upper Circumference
4W85-S-...	15 - 23 cm	23 - 35 cm
4W85-M-...	21 - 31 cm	29 - 44 cm
4W85-L-...	28 - 43 cm	36 - 52 cm
4W85-XL-...	34 - 48 cm	42 - 58 cm
4W85-XXL-...	38 - 52 cm	43 - 64 cm
4W85-XXXL-...	42 - 56 cm	50 - 70 cm


3.1.3 4W85

Measure circumference approx. 15 cm above and below mid-patella and choose the correct size.

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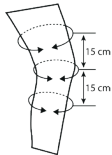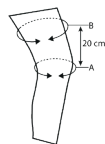
3.1.5 4W88/4W89

Measure circumference above mid-patella (A) and 20 cm proximally MPT (B) and choose the correct size.

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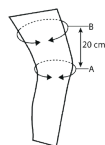4. Characteristics

- Thermoplastic deformable at approx. 60°C
- Offers high adhesion and great wearing comfort
- Gel cushions and minimizes shear forces
- additional rotational stability
- elastic outer fabric allows easy rolling up
- for use as vacuum and adhesive knee sleeve
- offers high security

5. Application (putting on and putting off)

- Remove the packaging foam from the knee sleeve and keep it for later reuse when storing the knee sleeve.
- The TPE knee sleeve should lie against the skin without wrinkles.
- Turn half of the knee sleeve outwards to expose the gel side. Pull the inverted half of the knee sleeve over the prosthesis.
- Climb into the prosthesis with liner (and, if using, residual limb sock and knee sleeve protection).
- Roll the sealing sleeve up over the knee. Avoid wrinkles, air pockets and soft tissue displacement as you do so

6. Storage

- Never let the knee sleeve dry on the heater or other direct heat sources.
- When not using the knee sleeve, put the packing foam back into the knee sleeve. This will prevent the formation of small wrinkles that can form in the gel if the knee sleeve is lying flat for a long time.
- Never store the knee sleeve inside-out.
- If you do not use the prosthesis for a longer period, please separate the knee sleeve from the socket.

7. Care instructions

Daily cleaning:

The TPE knee sleeve should be washed by hand after each use with a mild soap and warm water. Apply the soap either by hand or with a clean and soft washcloth or sponge. Please do not use a brush on the TPE knee sleeve. Scrubbing may roughen the surface of the knee sleeve, which may cause a skin reaction. Rinse off soap residues thoroughly and dry the knee sleeve with a dry and clean cloth.

Note!

Using the wrong detergent may damage the product.

Do not use bleach, harsh chemicals or objects such as salt water, acids, dust or sand as this may damage the fabric on the outside of the knee sleeve or the gel surface.

Weekly Disinfection:

Apply some ethyl or isopropyl alcohol to a soft and clean cloth. Now gently wipe the gel side of the knee sleeve with the alcohol-soaked cloth. Then dry the knee sleeve with a dry and clean cloth.

Note!

Do not dip or soak the TPE knee sleeve in ethyl or isopropyl alcohol, as prolonged contact with large amounts of ethyl or isopropyl alcohol will stiffen the liner. This will invalidate the warranty.

8. Maintenance/Service

The TPE knee sleeve is subject to natural wear and tear during continuous use and must therefore be checked at regular intervals and replaced if necessary.

9. Utilization

9.1 Restriction on Use

Due to the adjustment, which must be carried out by trained personnel, as well as for hygienic reasons, we would like to point out that this product may only be used by one person.

9.2 Misuse

The product is intended exclusively for prosthetic fitting in accordance with its intended use and is not suitable for other sports activities.

9.3 Service life

The product is a wearing part that is subject to normal wear and tear.

10. Guarantee

The following guarantee declaration applies in addition to any existing legal warranty claims and is independent of these.

10.1 Conditions of entitlement

The manufacturer provides a warranty on the product from the date of purchase.

A guarantee can only be given if the product is used under the specified conditions and for the intended purposes. The warranty covers defects that are demonstrably due to material, manufacturing or construction errors and that are claimed against the manufacturer within the warranty period. Claims under the manufacturer's warranty can only be made if the product concerned has been installed or adapted by or under the supervision of trained specialist personnel. Maintenance intervals, if any, must have been demonstrably carried out.

10.2 Scope and exclusion of warranty

The product is guaranteed to be free of material, manufacturing or construction defects. The manufacturer's warranty does not cover defects that are attributable :

- to incorrect installation
- on operating errors
- to improper / improper use
- on normal wear and tear
- on improperly machined or sharp-edged edges of the socket
- external influences such as force majeure or third party intervention

10.3 Duration of warranty

The duration of the manufacturer's warranty varies depending on the product. In any case, the duration of the manufacturer's warranty starts from the date of delivery to the customer. If used properly, the TPE knee sleeve is guaranteed for 3 months.

10.4 Liability

The manufacturer is only liable if the product is used in accordance with the descriptions and instructions in this document. The manufacturer is not liable for damage caused by non-compliance with this document, in particular by improper use or unauthorized modification of the product. Each manufacturer can only be held liable for the failure of its own fitting parts. A further liability of the manufacturer is only possible if his own components can be proven to have been the cause of the damage or functional failure of components from other manufacturers.

10.5 Warranty service

If all conditions for a warranty claim are met, Wagner Polymertechnik GmbH will, at its discretion, repair the damaged product, send a replacement free of charge or issue a credit note for the cost of the product.

11. Declaration of Conformity

As the manufacturer, Wagner Polymertechnik GmbH declares under sole responsibility that the LITE LINER® Gel sleeves comply with the requirements of Regulation (EU) 2017/745 for medical devices (MDR).

12. Disposal

The product is to be recycled or disposed of without risk to others or the environment. Please observe the applicable national regulations.

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