Purpose of the instructions

This Manual is relative to a dorso-lumbar brace, in compliance with Regulation 2017/745 concerning class 1 medical devices and intends to provide a guide for application, regulation (by orthopaedic specialist) and for safe and correct use (by patient) throughout all his life. Therefore the Declaration of Conformity has been issued by Tecnoway under its own responsibility in accordance with Annex IV of this Regulation.

Identification of the manufacturer and the product

Tecnoway produces medical devices, in particular orthopaedic braces. The firm is constantly searching for new materials innovative technologies and ergonomic solutions. Its system of quality management is certificated in accordance with ISO 9001 and ISO 13485 regulation. The product in this manual is a dorso-lumbar brace suitable for dorsal-lumbar fractures, spondylitis and spondylodiscitis, hyperkyphosis and dorsal kyphosis.

General description and mode of action

The brace in this manual is mainly made of an aluminium frame and supported by non-toxic material. This makes the device light, comfortable and safe. It reduces the deterioration of the sagittal trim of the spine and it relieves the pain, contributing to postural re-education without hampering the thoracic and abdominal breathing.

Caution for use

The device should be used exclusively for orthopaedic treatment of the trunk. The Orthopaedic specialist should decide on the time of daily duration. The pressure made by the device should not cause lesions or compress nerves, blood vessels. Always wear over garment. Some components of device are inflammable, don't use it near flames. Not use near strong electro-magnetic field. Be careful to open body-belt slowly to avoid a sudden collapse of the trunk.

Features

Combination structure: rigid back frame in aluminium alloy and front corset in elastic fabric. Non-toxic foam rubber padding with imitation-leather lining. Velcro fastening on the front of the corset. Shoulder straps with armpit padding and adjustable buckles. The body-belt is applied to the brace through zip so that it's easier to remove it in order to wash or replace it.

Indications

Fractures and the outcome of dorsal-lumbar fractures, osteomalacia fractures and osteoporosis. Spondylitis and spondylodiscitis, hyperkyphosis and dorsal kyphosis.

Application

1. The device must be applied in the back part of the trunk, the patient can lying or standing while doing this.
2. Place the band "A" on the sacral area of the patient (fig 1)
3. Adapt the brace to the morphology of the patient (manually or with help of tools) on the sacral band "A", the dorsal band "B", side frame "C" and spinal bars "E" (fig 1).
4. Tighten the body-belt "G" and fasten it with its Velcro (fig 3)
5. Pass belts "F" around the back under the armpits and thread buckle through loops, tighten belts.
6. The correct regulation verifies when:
 - the patient feels a well sustained lumbar pressure,
 - the straps are tightened so that the patient has adequately shoulders pushed back without feeling too much pressure on the armpits,
 - the spinal column is at the center of the two spinal bars, the sacral band at the coccyx level and the two spinal bars are at the shoulder blade level;
 - the shoulder pads are placed at the point of pressure exerted by belts,
 - the spinal bars, the sacral band, the dorsal band and side frames are fitting tightly to the body of the patient.

Warning

The device in question must be prescribed by an orthopaedic specialist and applied by an orthotist who is the competent figure of reference for the application and for the information to safe use. Any modification or regulation different to those placed on device at the moment of application should be prescribed by an orthopaedic specialist both for application and safe use. The application must be done with maximum care, to assure effectiveness, tolerability and correct use. Don't change the regulation done by the orthopaedic specialist. Any structural changes or other must be decided by the orthopaedic specialist and performed by the orthotist. For hygienic and functional reason we recommend the use for single patient because it could cause risks for hygienic or functional aspect. In particular sensible person, direct contact of skin to device may cause redness and/or irritations, if those problems may occur pls. contact directly your doctor. If the device is no more under normal use it should be replaced and disposed off.

Incident reporting notice

Any serious incident that occurs in connection with this medical device must be reported to the manufacturer and the competent authority.

Maintenance

Before use always check the condition of the components. If worn out contact your orthopaedic specialist to substitute. Hand wash the device with warm water and mild soap. Rinse accurately. Dry away from heat sources.

Storage instructions

When not using the device, even for short periods, store it in its original packaging to maintain a good level of cleanliness. Keep it in a cool, dry place and protect from direct sunlight.

Disposal

Make sure that the device is disposed off in a correct manner. Do not disperse in the environment.

Guarantee

The guarantee is valid only if the product is used under the conditions indicated and under the intended purpose.

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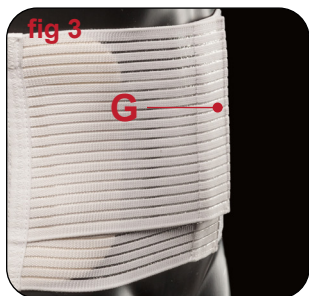
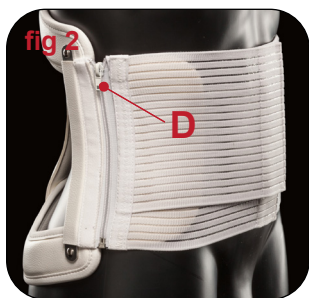
This manual has been realized by Tecnoway exclusively for its clients. Its reproduction and/or entirely and/or partial divulgation is forbidden without Tecnoway authorization. This manual comes together with equipment device, and should be kept in an easy access and suitable place. It should be preserved for all the device life cycle. Personal assigned to application, regulation, use and maintenance should carefully read and understand the instructions of this manual. Tecnoway cannot be pursued or held responsible for any damage caused by wrong application different to those in this manual.

 by **TECNOWAY SRL**
via Montefeltro, 45 - 61022 Vallefoglia -PU- ITALY
+39.721.496175 info@tecnoway.it www.tecnoway.it

COMPANY WITH SYSTEM CERTIFICATION
ISO 9001 - ISO 13485 - ISO 45001

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